# Refine One Step

## Table of Contents

EN..... 1  
 BG..... 4  
 CS..... 6  
 DA..... 8  
 DE..... 10  
 EL..... 12  
 ES..... 14  
 ET..... 16  
 FI..... 18  
 FR..... 20  
 HU..... 22  
 IT..... 24  
 NL..... 26  
 NO..... 28  
 PL..... 30  
 PT..... 32  
 RU..... 33  
 SK..... 36  
 SL..... 38  
 SV..... 40  
 TR..... 42

 0123	 CooperVision Hamble, SO31 4RF, UK <a href="http://www.coopervision.com">www.coopervision.com</a>	 CooperVision CL Kft. Gorcsev Iván street 7. Building C. 2360 Gyál, Hungary	 25°C				


## EN

### Instructions for Use

#### IMPORTANT

Read Instructions carefully before use

**Figure 1:**

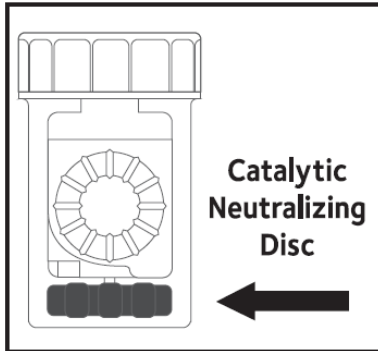### PRESERVATIVE-FREE ONE STEP PEROXIDE SOLUTION

For use with all contact lenses including silicone hydrogel.

#### Indications for use

Cleans, disinfects, lubricates, stores and removes protein. For use with all contact lenses including silicone hydrogel.

Contains a cleaning agent. NOT TO BE USED FOR RINSING.

THE VAST MAJORITY OF PROBLEMS ASSOCIATED WITH CONTACT LENS WEAR ARE CAUSED BY FAILURE TO FOLLOW THE CONTACT LENS CARE INSTRUCTIONS EXACTLY.

Intended users: For all contact lens wearers including silicone hydrogel.

Contra-indications: Do not use if allergic to any ingredient.

'Preservative-free one step peroxide solution' cleans, disinfects, lubricates and stores all contact lenses in one easy step. It consists of a gentle surfactant cleaner as an integral part of the hydrogen peroxide solution used in conjunction with the special catalytic disc barrel case provided.

#### IMPORTANT:

- Do not change from the solutions recommended by your contact lens practitioner.
- Do not use a soaking case that does not contain the neutralizing disc (SEE FIGURE 1.).

#### INSTRUCTIONS FOR USE:

1. Wash, rinse and dry your hands thoroughly before handling your lenses. 2. Open the basket of your barrel case marked 'R' and place your right lens on the dome of the central holder. Ensure the lens does not overlap the spokes of the holder and does not become pinched when the basket is closed. Close the basket. 3. Repeat with your left lens in the other basket. 4. Fill the barrel case up to the marked line with your 'preservative-free one step peroxide solution'. Do not over fill the lens case. It is important that you do not pour your 'preservative-free one step peroxide solution' into the barrel case until your lenses have been placed into each basket. 5. Place the lens holder into the barrel case and close securely, taking care not to over tighten the cap. 6. Allow your lenses to soak for at least 6 hours or overnight ensuring that the barrel case is in an upright position and that the lenses are fully immersed in the 'preservative-free one step peroxide solution'. 7. After a period of at least 6 hours, your lenses

may be removed from the barrel case and worn. 8. Always discard the remaining solution from the barrel case. 9. Rinse the barrel case thoroughly with sterile saline and leave it open to air dry. If lenses are to be worn for extensive periods without replacement, a separate daily cleaner may be required. Consult your contact lens practitioner. The weekly use of protein removal tablets may be necessary for some patients. Please consult your contact lens practitioner.

**WARNINGS:**

- Do not apply your 'preservative-free one step peroxide solution' directly into the eye.
- Important: Do not over tighten the lens case cap.
- Do not wear the lenses unless you have allowed them to soak for at least 6 hours in your 'preservative-free one step peroxide solution'. If not completely neutralised, hydrogen peroxide solution can cause a stinging sensation. If this happens, remove lenses from eyes immediately and thoroughly rinse your eyes with running water.
- Not to be used for rinsing contact lenses.
- Keep out of reach of children.
- Do not use after the expiry date indicated on the packaging.
- Do not rinse contact lenses or your contact lens case with water directly from the tap.

**PRECAUTIONS:**

- Do not mix with other fluids except as directed.
- Do not substitute products from your 'preservative-free one step peroxide solution' system with products from other systems, even if hydrogen peroxide based.
- In the case of persistent eye irritation discontinue use immediately, remove lens from eye and consult your contact lens practitioner.
- Discard contents of bottle 3 months after first opening.
- Store below 25°C.
- Always use a new barrel case and catalytic disc with each new bottle of solution.
- Always discard the remaining solution from the lens case. Never re-use.
- If lenses are stored for more than seven days in your 'preservative-free one step peroxide solution', disinfect your lenses again by replacing the solution in the barrel case with fresh 'preservative-free one step peroxide solution'.
- Do not over fill the barrel case.
- Close bottle securely after use.
- Do not use this product if you experience an allergic reaction.
- Do not allow the nozzle tip to touch any surface.
- During disinfection and storage of lenses, keep lens case in an upright position and in a safe place.
-  Do not use if tamper-evident seal is damaged.

Any serious incident that has occurred in relation to the product should be reported to the manufacturer and applicable competent authority.

The sterile barrier for solution is the unopened bottle cap.

Disposal: use standard waste, recycle whenever possible.

**INGREDIENTS:** 3% Hydrogen peroxide, sodium chloride, sodium phosphates, poloxamer, HEDP, water.

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